Record your model number:

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Anoto su número de modelo: _____

CAUTION: Risk of product damage. To avoid cross-threading, start threading the armrest bolt by hand then continue using a Phillips screwdriver.

Follow all local plumbing and building codes.

Verify that your toilet is properly secured to the floor.

Armrest Assembly: Steps 1-6

5A. Slide the top cover onto the armrest.

5B. Rotate the top cover and align into position.

6A. The right and left side of the armrest have specific hole positions. Please verify you have the correct bottom plate for each side.

Armrest and Toilet Seat Installation: Steps 7-14

8A. Fold the anchor as shown.

9A. Insert the folded anchors into the two toilet bowl holes.

9B. Verify that the anchor unfolds completely when installed.

12A. To avoid cross-threading, start threading the armrest bolt by hand then continue using a Phillips screwdriver. **To allow for adjustment of the seat and armrest, do not fully tighten the armrest bolts.** To fully compress the anchors to the toilet, many rotations with the Phillips screwdriver may be required.

13A. Lift the arms to verify that the arms clear the toilet tank and trip lever. Align the seat to the toilet bowl. Adjust if necessary.

14A. Firmly tighten the two screws. Verify that the armrest and seat are secured to the toilet bowl. There should be no movement of the seat and armrest.

Need help? Contact our Customer Care Center.

• USA/Canada: 1-800-4KOHLER (1-800-456-4537) Mexico: 001-800-456-4537

• **Service parts:** kohler.com/serviceparts

• **Care and cleaning:** kohler.com/clean

• **Patents:** kohlercompany.com/patents

This product is covered under the **KOHLER® One-Year Limited Warranty**, found at kohler.com/warranty. For a hardcopy of warranty terms, contact the Customer Care Center.

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