

DEKODA

QUICK START GUIDE

# Download the Kohler Health app

## 01 Download the Kohler Health app from the App Store or Google Play.




The Kohler Health app requires Bluetooth and WiFi connection to use with your Dekoda device.



## 02 Log in or create your Kohler Health account.

## 03 Follow the in-app instructions to set up and install your Dekoda device.

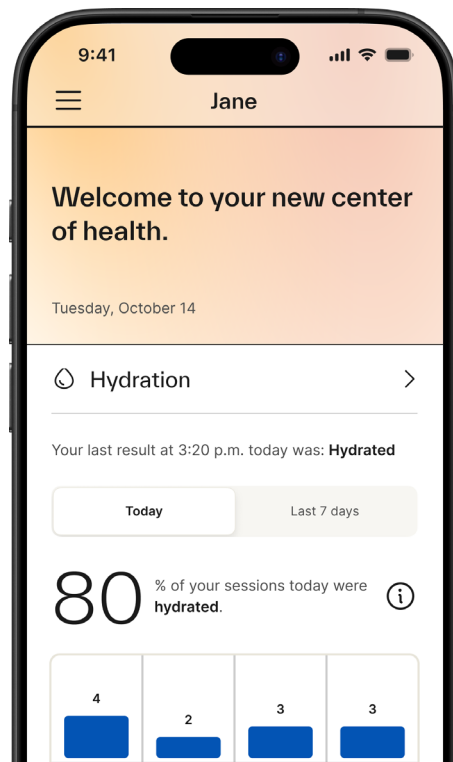

# About your Battery

Your Dekoda comes with a partially charged battery. **Please fully charge the battery before starting the installation.**

If you have an outlet near your toilet, we recommend keeping Dekoda plugged in for the best experience.

| Color   | Solid Light   | Blinking Light |
|---------|---------------|----------------|
| ● Green | Fully charged | Charging       |



# Get to Know your Dekoda



Unsure about toilet fit?

Scan to view our compatibility guide. Email us at [support@kohlerhealth.com](mailto:support@kohlerhealth.com) for assistance.

## Status light

| Color          | Solid Light | Blinking Light      |
|----------------|-------------|---------------------|
| ○ White        | Ready       | Waking up           |
| ● Purple       | Battery low |                     |
| ● Blue         |             | Pairing mode        |
| ● Red          | Error       | Wi-Fi disconnected  |
| ● Blue & Green |             | Firmware installing |

## Power button

Press to turn Dekoda on/off.

## Proximity sensor

Automatically wakes the device on motion detection.

## Adjustable clamp

Adjusts to fit your toilet.





Do not spray cleaners directly onto your device. Use only cleaning wipes for best results.

## Battery

Can be charged while attached or detached from the device.

### Battery status light

| Color   | Solid Light   | Blinking Light |
|---------|---------------|----------------|
| ● Green | Fully charged | Charging       |

### USB-C charging port

For use with the included power cable and charging adapter.



Sensors

### Magnetic pad

Pivots for stability.  
Remove only to clean.

# Get to Know your Remote

## Primary button


Confirm an action or scan your fingerprint to start a session.

## Status light

| Color   | Status                 |
|---------|------------------------|
| ○ White | Ready to scan          |
| ● Blue  | Pairing mode           |
| ● Green | Fingerprint recognized |
| ● Red   | Can't read fingerprint |

## AA battery compartment

Slide to open.

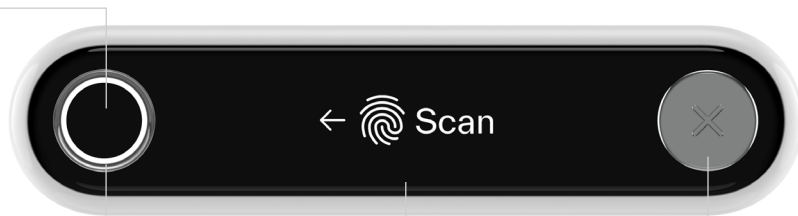

## Screen

Follow on-screen messaging as needed.

## Cancel button

Cancel any ongoing activity.



# Included Accessories

---

## Wall-mount remote holder

An optional remote holder that mounts on the wall near your toilet for convenient one-touch activation.



|                |               |                         |                                               |
|----------------|---------------|-------------------------|-----------------------------------------------|
| Charging cable | Power adapter | AA batteries for remote | Remote holder screws,<br>anchors and 3M™ tape |
|----------------|---------------|-------------------------|-----------------------------------------------|



For more detailed information or help,  
please scan the QR code to visit our support page.

**KOHLER**® Health