BEFORE YOU BEGIN

- This grab bar is used to assist people to stand near to the basin.
- To ensure correct protection and assembly of this product, please read this manual carefully before installation.
- Please keep this manual for reference.
- Information herein is selected from the published manual of this product. KOHLER reserves the right to change the specification, packing, or types of this product without prior notice.
- This grab bar meets requirements of the standard GB/T 22752-2021.

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PRECAUTIONS

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Before installation:

- Please ensure that the countertop where the grab bar is installed has sufficient strength to prevent the grab bar from loosening and accidents.
- If sensor products are installed beside the grab bar, please install such products in the place where the grab bar cannot interfere.
- Please use the self-tapping screw and anchor that comes with the grab bar to lock the grab bar.

After installation:

- Please try it in time to ensure that the grab bar functions normally and there is no looseness.
- To avoid accidents:
 1. Do not use grab bar with wet hands, or hands with detergent or lubricating oil.
 2. Do not climb the grab bar or reach into the area between the grab bar and the countertop.
 3. Do not place objects on the grab bar or hang heavy objects.
 4. Do not shake the grab bar on purpose frequently.
- Please check regularly whether the grab bar are loose.
- Do not intentionally scratch or touch the surface of the grab bar to avoid damage to the surface.

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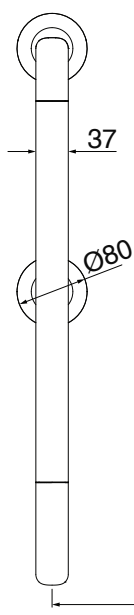
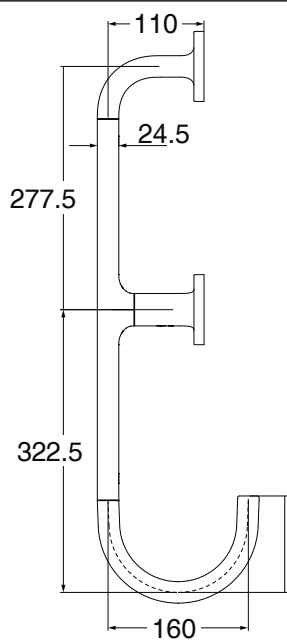
*-Recommended dimension

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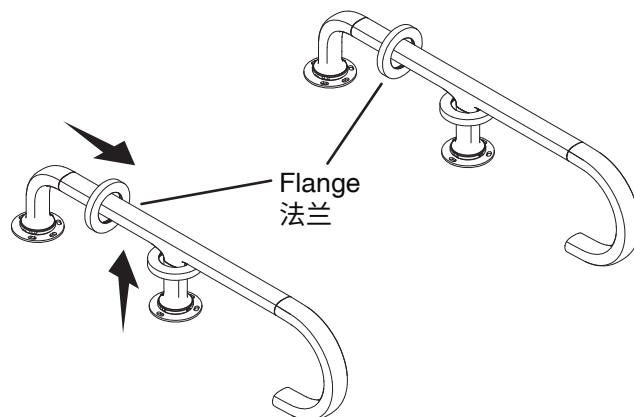
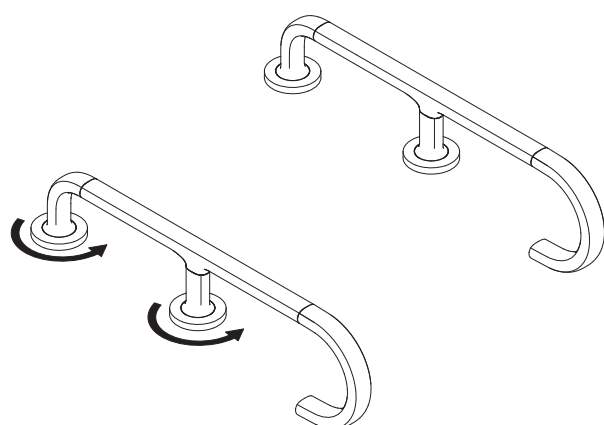
INSTALLATION

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

Rotate the flange counterclockwise to unscrew and move away from the base.

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
Place the both grab bar at recommended distance 300 mm from the center of basin or faucet, and make sure the free end of both grab bar is close or touch the front surface of vanity. Mark the hole position for drilling.

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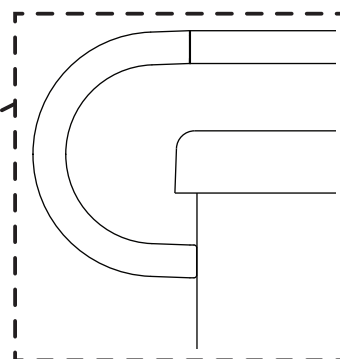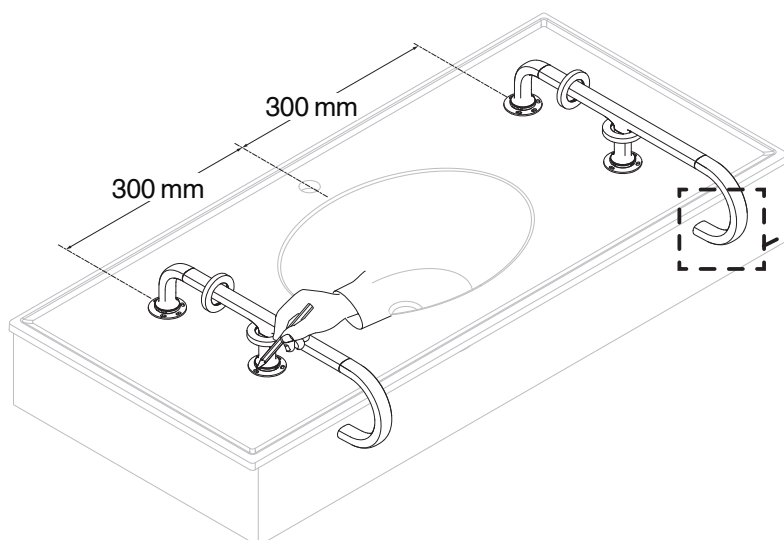Drill twelve installation holes with Ø8 mm diameter and upto the thickness of countertop according to the marked points.

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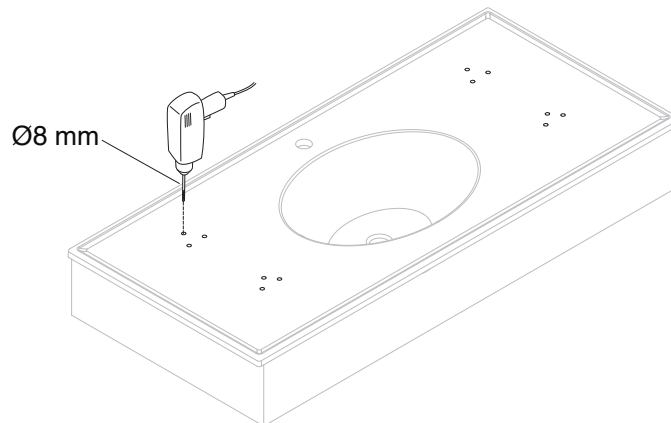Insert the anchor into the drilled hole until its end surface does not protrude from the countertop. Pass the self-tapping screw through the mounting hole of the base and tap into the anchor to lock the grab bar on the countertop.

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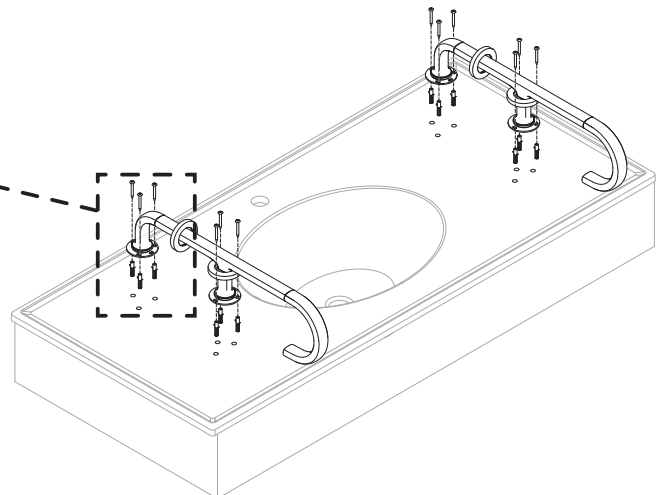
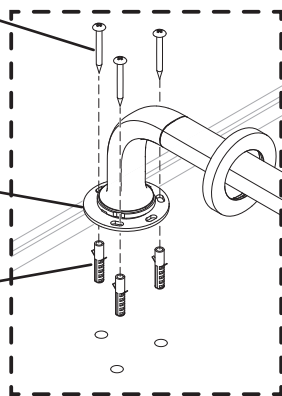
Note: The self-tapping screw must be locked to the base.

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Base
底座

Anchor
膨胀管

Rotate the flange clockwise to lock it on the grab bar base.

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