BEFORE YOU BEGIN

- This grab bar is used to assist people to stand up from the toilet or in other places.
- To ensure correct protection and assembly of this product, please read this manual carefully before installation.
- Please keep this manual for reference.
- Information herein is selected from the published manual of this product. KOHLER reserves the right to change the specification, packing, or types of this product without prior notice.
- This grab bar meets requirements of the standard GB/T 22752-2021.

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PRECAUTIONS

Before installation:

- Please ensure that the wall where the grab bar is installed has sufficient strength to prevent the grab bar from loosening and accidents.
- If sensor products are installed beside the grab bar, please install such products in the place where the grab bar cannot interfere.
- Please use the self-tapping screw and anchor that comes with the grab bar to lock the grab bar.

After installation:

- Please try it in time to ensure that the grab bar functions normally and there is no looseness.
- To avoid accidents:
 1. Do not use grab bar with wet hands, or hands with detergent or lubricating oil.
 2. Do not climb the grab bar or reach into the area between the grab bar and the wall.
 3. Do not place objects on the grab bar or hang heavy objects.
 4. Please do not shake the grab bar on purpose frequently.
- Please check regularly whether the grab bar are loose.
- Do not intentionally scratch or touch the surface of the grab bar to avoid damage to the surface.

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
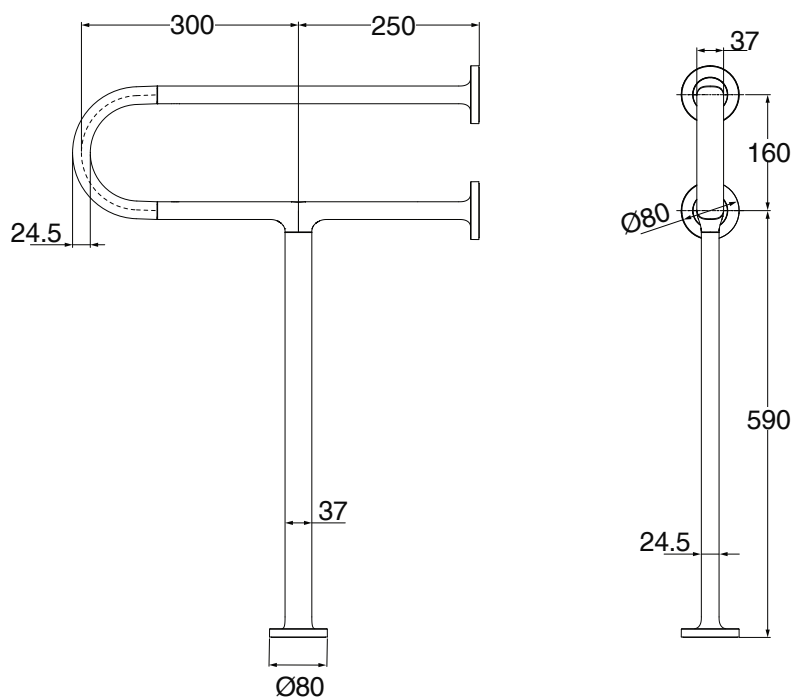
K-30765T-ST


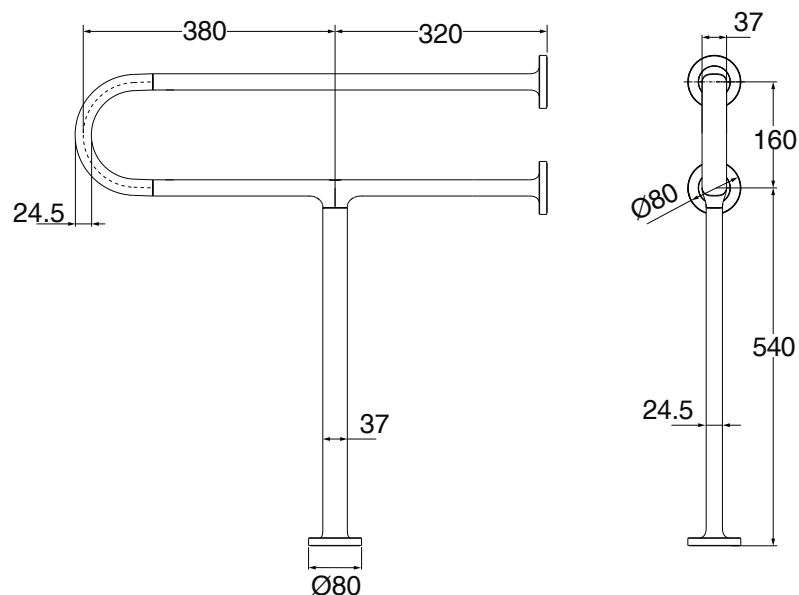
INSTALLATION

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
Align the vertical grab bar with U shaped grab bar and lock with screw. once the both grab bars are locked then install the stopper on screw hole.

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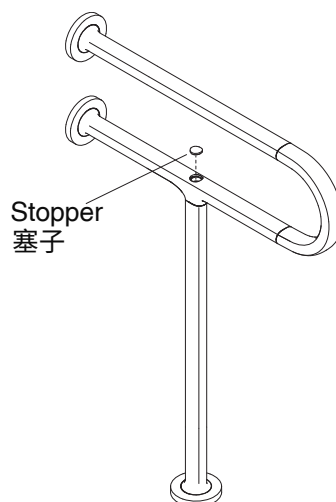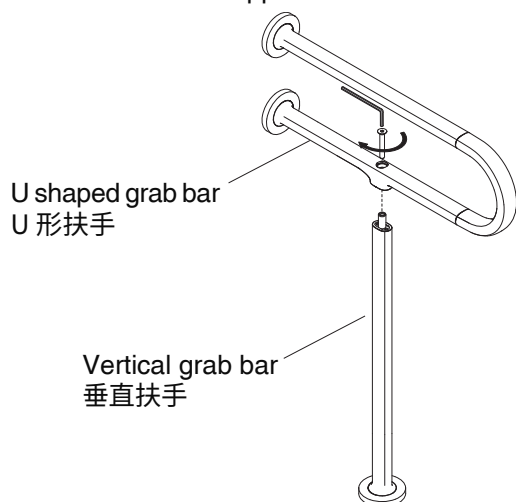
Rotate the flange counterclockwise to unscrew and move away from the base.

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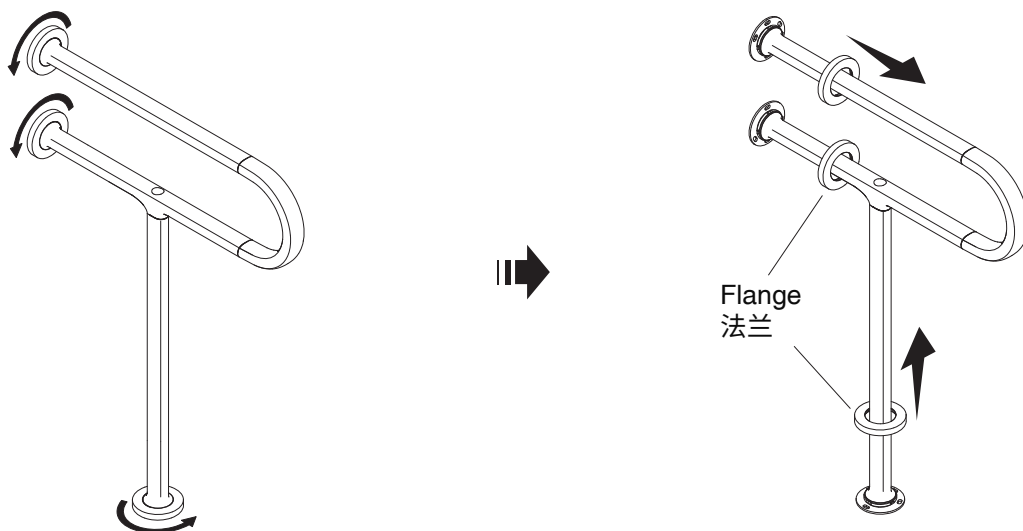
Mark the hole position for drilling.

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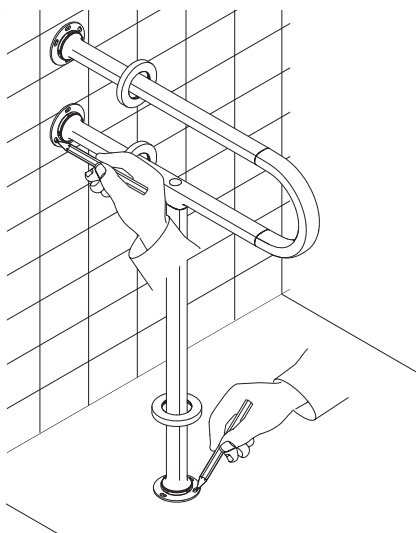Drill nine installation holes with Ø8 mm diameter and about 50 mm depth on the wall and floor according to the marked points.

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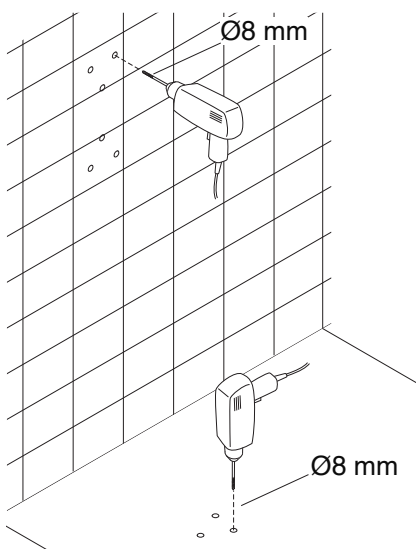
Insert the anchor into the drilled hole until its end surface does not protrude from the wall and floor. Pass the self-tapping screw through the mounting hole of the base and tap into the anchor to lock the grab bar on the wall and floor.

Note: The self-tapping screw must be locked to the base.

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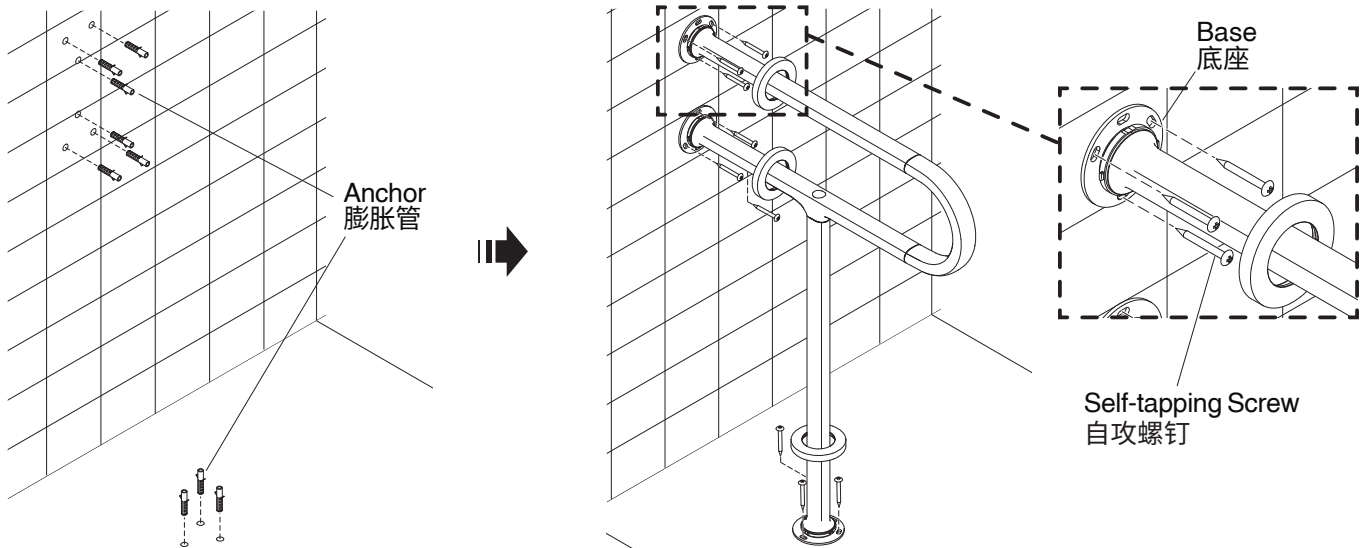
Rotate the flange clockwise to lock it on the grab bar base.

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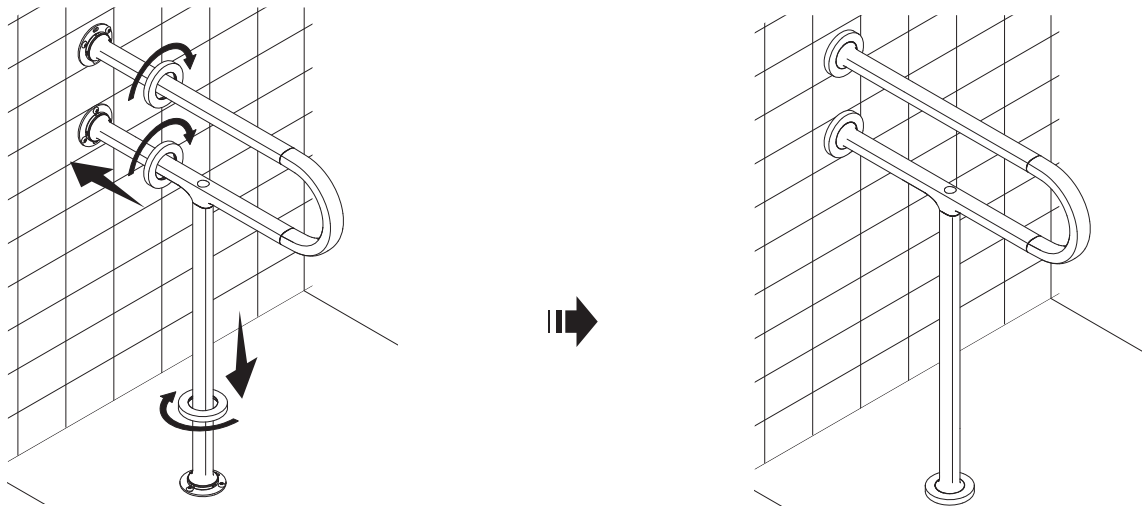CLEANSING AND MAINTENANCE

- Please use mild cleaners (such as liquid cleaners and warm water) to clean the product. Do not use abrasive cleaning products or concentrated cleaners or the surface of this product may be damaged or this product may tarnish.
- Please read the instructions of cleaners carefully to ensure that cleaners are suitable for this product.
- Before cleaning, please try to use cleaners to clean an inconspicuous part of the product.
- After wiping the product with cleaners, please immediately wipe and rinse the surface with water. At the same time, wipe cleaners splashed around.
- Please use soft and wet sponge or cloth. Do not use abrasive cleaning materials such as brushes or scouring pads.
- If the grab bar is installed in the shower room, rinse the surface of the grab bar with clean water in time after taking a shower. Then dry the surface with a soft cloth so that it is kept clean and hygienic.

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